iHealth®

Early Pregnancy Test Instructions for Use

Before You Begin

PLEASE CAREFULLY READ THE INSTRUCTIONS FOR USE BEFORE TESTING

- Home pregnancy test. For in vitro diagnostic use only.
- Do not open the foil pouch until ready. It must be used within one hour of opening.
- Store at **36-86 °F (2-30 °C)**. Do not freeze.
- Have a timer available. (Optional: Have a urine sample collection cup available.)

Step-by-Step Instructions

1 Testing

Tear open the foil pouch, take out the test stick, and remove the cap. Holding the test stick by the finger grip, test using EITHER method:

Note: Ensure the cap is secure on the finger grip end of the test stick.

Method 1

2-3 seconds

Tilt the absorbent tip downward with the result window facing you. Hold the tip in your urine stream for **2 to 3 seconds**.

Method 2

10 seconds

Use a clean, dry container (not provided with this kit) to collect a urine sample. Dip the absorbent tip of the test into the urine sample for **10 seconds**.

Note: Make sure that the absorbent tip is completely submerged in the urine sample.


After testing with either method, place the test stick on a flat surface with the result window facing up.

2 Wait 3 to 5 Minutes


Please check the test result after 3 minutes, but do not read the test result after 5 minutes.

3-5 minutes

3 Read Your Result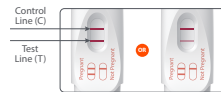

PREGNANT:
Two lines appear in the result window. The test line may be faint.

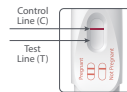NOT PREGNANT:
Only the control line appears in the result window.

TEST ERRORS (Invalid):
No line appears or the control line fails to appear. Re-test with a new test; do not reuse the test.

Frequently Asked Questions (FAQ)

Q: How does the iHealth® Early Pregnancy Test work?

A: The iHealth® Early Pregnancy Test checks for the presence of the hormone hCG (human chorionic gonadotropin) in your urine. This hormone usually increases to a detectable level in urine a few days after conception. Drinking too much water may dilute the concentration of hCG in urine.

Q: When can I test with the iHealth® Early Pregnancy Test?

A: You can use the iHealth® Early Pregnancy Test as early as 6 days before your missed period (5 days before you expect your period). The amount of hCG in the urine increases as a pregnancy progresses. If you test early and the result is negative (not pregnant), there is still a chance that you might be pregnant. In clinical testing with early pregnancy samples, the iHealth® Early Pregnancy Test gave the following results:

NUMBER OF DAYS BEFORE THE EXPECTED PERIOD	1	2	3	4	5
% OF SAMPLES FROM PREGNANT WOMEN GIVING A POSITIVE RESULT	>99%	99%	99%	93%	78%

Q: What time of day should I take a test?

A: The first morning urine has the highest hCG concentration but samples any time during the day can be used.

Q: What should I do if the test result is PREGNANT?

A: A positive pregnancy test result indicates you may be pregnant. If your result is positive, you should see your doctor. A confirmed pregnancy diagnosis should only be made by a physician after all clinical and laboratory findings have been evaluated.

Q: What should I do if the test result is NOT PREGNANT?

A: When the test result is negative, it indicates that you may not be pregnant. If pregnancy is still suspected, test again using a new device. If you still get a negative result and your period is late, you should see your doctor.

Q: When should I read the test result?

A: The test result should be read as soon as possible after waiting for 3 minutes. Test results should not be read after 5 minutes.

Q: Will taking certain medications affect my test result?

A: Always read the manufacturers' instructions for any medication you are taking before testing. If you get any unexpected results you should discuss them with your doctor. Medications containing hCG and other hormone therapies can affect the result. The test should NOT be affected by common painkillers, alcohol, or antibiotics.

Intended Use

The iHealth® Early Pregnancy Test is an OTC in vitro diagnostic test used to assess early pregnancy by detecting the presence of the hCG (human chorionic gonadotropin) hormone in urine. The test device is intended for use as an aid in early detection of pregnancy, in some cases as early as six (6) days before the day of the missed period, i.e. as early as five (5) days before the day of the expected period.

Warnings and Precautions

- Keep out of reach of children.
- Contains small parts. Do not ingest any components.
- Test components are for single use only. Do not reuse.
- Do not use the test after the expiration date marked on the package.
- All results should be confirmed by your healthcare provider, especially when making decisions about future medical care.
- Do not use the test if the foil package is damaged.
- Discard with normal household waste.

The iHealth® Early Pregnancy Test is >99% accurate from the day of the expected period in clinical data.

If you have any questions about the instructions or your results, call our Helpline at 1-855-816-7705 (8:30 AM - 5:30 PM PST, Monday to Friday except holidays).

For other updated FAQ information, please visit our website: <https://www.ihealthlabs.com>

Manufactured for iHealth Labs, Inc. 880 W Maude Ave, Sunnyvale, CA 94085 USA

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Fabricado por iHealth Labs, Inc. 880 W Maude Ave, Sunnyvale, CA 94085 USA