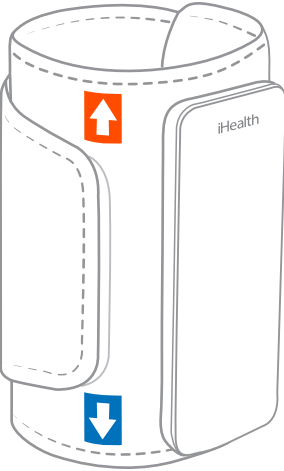
# iHealth Neo

## Quick Start Guide

### Activate the monitor

- 1 Press and hold START button at least 3 seconds until the LED screen displays all characters to activate the monitor.

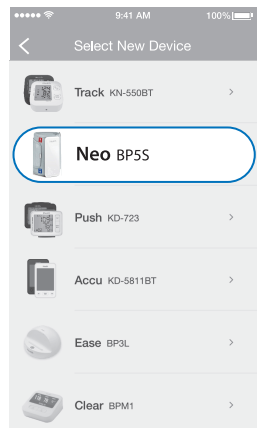
 If the monitor fails to activate, please make sure it is fully charged.

### Connect to iHealth MyVitals

- 2 Scan the QR code below to download and install the iHealth MyVitals app. Follow the on-screen instructions to register and set up your account.


- 3 After setting up your account, tap the "+" on the top right corner, then select "Neo (BP55)" in the list. Follow the steps on the screen to complete the connecting process.


### Measurement

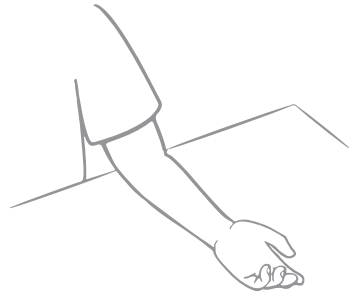
- 1 Rest your bare (or thinly clothed) left arm on a table.

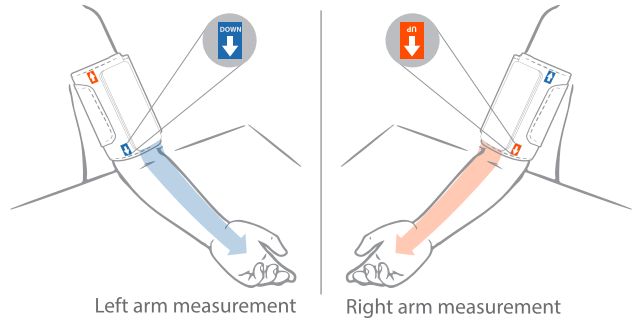
- 2 When measuring blood pressure on the left arm, ensure the blue arrow  is pointing downward. When measuring blood pressure on the right arm, ensure the red arrow  is pointing downward. Place the monitor on the middle of your upper arm, aligning it with your palm when it's facing up.

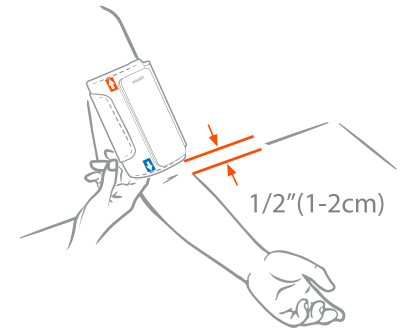- 3 Place the cuff 1/2" (1-2cm) above your elbow joint. Secure it and leave enough space to fit one finger between the cuff and your arm.

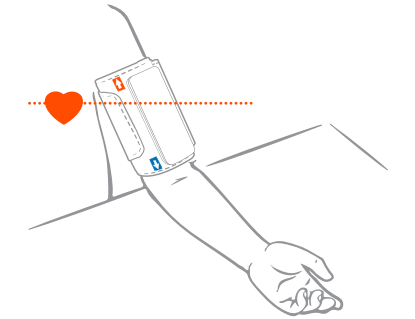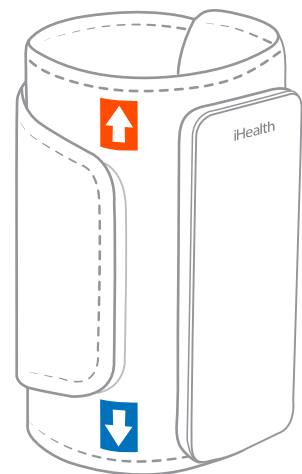
- 4 Lastly, ensure the middle of the cuff is at the same level as your heart, and press the START button to take the measurement.

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