

# iHealth<sup>®</sup> Track Pro

## Connected Blood Pressure Monitor Instruction Manual

MODEL **KN-550LT**





# TABLE OF CONTENTS

IMPORTANT INFORMATION	1
DISPLAY INDICATORS	1
BOX CONTENTS	2
INTENDED USE	2
SAFETY SYMBOLS USED IN THIS INSTRUCTION MANUAL	2
CONTRAINDICATION	2
PRODUCT DESCRIPTION	3
SPECIFICATIONS	3
SETUP AND OPERATION	4
1. DOWNLOAD THE APP (Recommend)	4
2. INSTALLING BATTERY AND AC ADAPTER LOADING	5
3. TIME AND DATE SETUP	7
4. CONNECTING THE CUFF TO THE DEVICE	9
5. APPLYING THE CUFF	9
6. BODY POSTURE DURING MEASUREMENT	11
7. TAKING YOUR BLOOD PRESSURE READING	12
8. VIEWING STORED READINGS	14
9. TRANSFERRING STORED READINGS TO THE APP	16
IMPORTANT SAFETY INFORMATION	17
ASSESSING HIGH BLOOD PRESSURE FOR ADULTS	20
NORMAL BLOOD PRESSURE FLUCTUATION	21
TECHNICAL ALARM DESCRIPTION	22
TROUBLESHOOTING	22
MAINTENANCE	24
EXPLANATION OF SYMBOLS ON UNIT	25
WARRANTY INFORMATION	26
ELECTROMAGNETIC COMPATIBILITY INFORMATION	27


## IMPORTANT INFORMATION

Please read this instruction manual carefully before using the product. Thank you for purchasing the iHealth Track Pro Connected Blood Pressure Monitor. Please retain this Instruction Manual for reference.

## DISPLAY INDICATORS

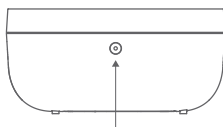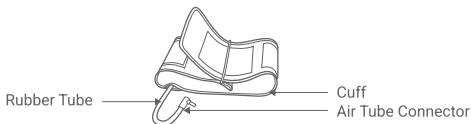



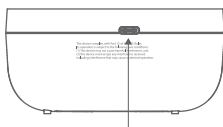




Air Tube Socket



USB Type-C External Power Connector

## The explanation of symbols on display:

- |  |  |
|--|--|
|  Low battery indicator |  Movement symbol            |
|  Bluetooth symbol      |  Cuff wrap loose symbol     |
|  Memory indicator      |  Irregular heartbeat symbol |
|  Heartbeat symbol      |  |



## BOX CONTENTS

- 1 × Blood Pressure Monitor
- 1 × Instruction Manual
- 1 × Cuff (8.7" - 16.5")
- 3 × AA Batteries
- 1 × USB Type-C Power Cable

## INTENDED USE

Thank you very much for selecting iHealth Track Pro Connected Blood Pressure Monitor. This monitor is for medical professionals or individuals (patients) at home. It is a non-invasive blood pressure measurement system intended to measure the diastolic blood pressure, systolic blood pressure, and pulse rate of an adult individual using a noninvasive technique in which an inflatable cuff is wrapped around the arm. The cuff circumference is limited to 5.9"-18.9" (15-48cm).

## SAFETY SYMBOLS USED IN THIS INSTRUCTION MANUAL

 Warning	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
 Caution	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user.

## CONTRAINDICATION

- △ This blood pressure monitor (electronic sphygmomanometer) is not suitable for people with severe arrhythmia.

## PRODUCT DESCRIPTION

The iHealth Track Pro is a fully automatic arm-cuff blood pressure monitor that uses the oscillometric methodology to measure your blood pressure and pulse rate. The monitor works with your mobile devices to track and share vital blood pressure data.

This blood pressure monitor has been designed in accordance with the requirements of ISO 81060-2:2018.

## SPECIFICATIONS

1. Product name: iHealth Track Pro Connected Blood Pressure Monitor
2. Model: KN-550LT
3. Classification: Internally powered, Class II, Type BF applied part, IP21, No AP or APG, continuous operation, not intended for use in an OXYGEN-RICH ENVIRONMENT.
4. Monitor size: Approx. 3.9" × 3.9" × 1.8" (98 mm × 98 mm × 46 mm)
5. Cuff circumference: 8.7" - 16.5" (22 cm - 42 cm).
6. Weight: Approx. 7.16 oz. (203g) (excluding batteries and cuff)
7. Measuring method: Oscillometric method
8. Memory Storage: 120 measurements
9. Power source:  
Medical AC adapter: 5V  $\equiv$  1A, batteries: 3 × 1.5V  $\equiv$  SIZE AA
10. Measurement ranges:  
Cuff pressure: 0 to 300 mmHg  
Systolic: 60 to 260 mmHg  
Diastolic: 40 to 199 mmHg  
Pulse rate: 40 to 180 beats/minute
11. Accuracy:  
Pressure:  $\pm 3$  mmHg  
Pulse rate: Less than 60:  $\pm 3$  bpm  
More than 60 (incl.):  $\pm 5\%$   
The precision of the displayed values: 1 mmHg
12. Environmental temperature for operation: 41°F to 104°F (5°C to 40°C)
13. Environmental humidity for operation:  $\leq 85\%$  RH

14. Environmental temperature for storage and transport: -4°F to 131 °F (-20°C to 55°C)
15. Environmental humidity for storage and transport: ≤90% RH
16. Environmental pressure: 80 kPa to 105 kPa
17. Battery life: Approx. 450 measurements
18. Wireless communication:
  - Bluetooth 5.3
  - Modulation types: GFSK
  - Frequency Band: 2.400 GHz to 2.4835 GHz
  - Effective radiated power: < 0 dBm
19. Product life:
  - Monitor: 5 years
  - Cuff: 3 years

## SETUP AND OPERATION

### 1 DOWNLOAD THE APP (Recommend)

---

Before first use, download and install the iHealth MyVitals app from the App Store (iOS device) or Google Play (Android device). Scan the QR code or use the search term “myvitals”.




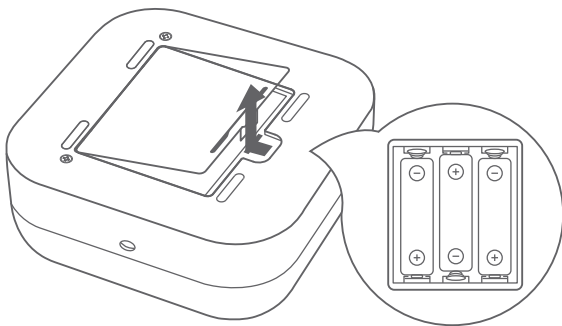
## 2 INSTALLING BATTERY AND AC ADAPTER LOADING

### INSTALLING BATTERY

1. Open the battery cover at the back of the device.
2. Insert three "AA" batteries. Make sure the batteries are inserted according to the positive and negative marks ("+" and "-") printed in the battery housing.
3. Close the battery cover.


Note:

- When a low battery symbol  shows on display, replace all batteries with new ones.
- Rechargeable batteries are not suitable for this device.



## AC ADAPTER LOADING

1. Insert the USB Type-C Power Cable on the right side of your monitor and the AC adapter.
2. Plug the AC adapter into an electrical outlet.

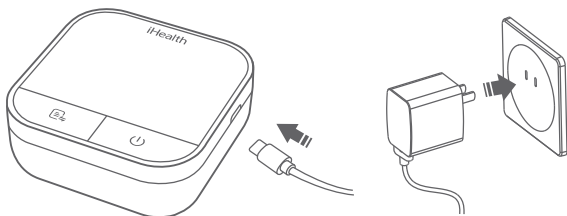

Note:


- The device automatically switches to external power when connected and reverts to battery power when disconnected.
- The external power source does not charge the device's battery.

---

To ensure every reading has the correct time and date, it is recommended to keep the **battery installed in the device at all times**.

Without keeping the battery in your monitor, you may need to set the date and time each time you unplug and plug back in the AC adapter.


- **⚠** Avoid getting battery fluid in your eyes. If battery fluid gets in your eyes, immediately rinse with plenty of clean water and consult with your physician.

- ⚠ The negative (-) side of the battery should be touching the spring.
- ⚠ Ensure the battery cover is intact and not damaged before installing the battery.
- ⚠ When connecting the adapter, connect to the device first and then the AC outlet. When disconnecting the adapter, disconnect the AC outlet first and then the device.
- ⚠ Do not plug or unplug the power cord into the electrical outlet with wet hands.
- ⚠ When using AC adaptor, make sure the device is turned off and that there are no batteries.
- ⚠ Do not overload power outlets. Plug the device into the appropriate voltage outlet.
- ⚠ If the AC adapter is abnormal, please stop using it.
- ⚠ Do not pull out the adapter while you are using the device.
- ⚠ Do not position the device so it is difficult to operate or disconnect.
-  The device, batteries, power cable, and cuff must be disposed of according to local regulations after use.





### 3 TIME AND DATE SETUP

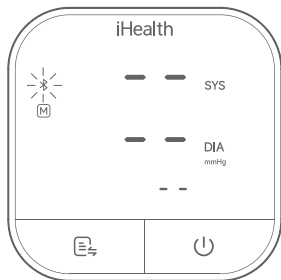
#### Automatic Time Sync (Recommend)

When the device connects to the app via Bluetooth, time and date will be synchronized automatically. It is recommended to set up the time before the first measurement.



## Manual Time Setting

1. Press and hold both  and  buttons for 2s to enter setting mode.
2. For each flashing item, press to  adjust and  to confirm. The setting order is “Year/Month/Day/Hour/Minute.”

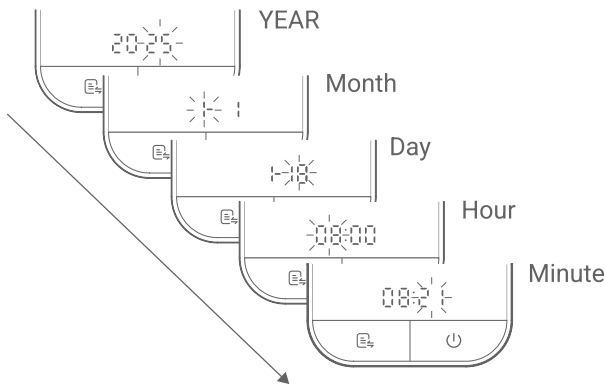

### Note:

- The device will exit setting mode after 30 seconds of inactivity.
- After replacing the batteries, you should sync/readjust the time and date.


## 4 CONNECTING THE CUFF TO THE DEVICE

---

Insert the air tube connector firmly into the device's air tube socket on the left side.

Note:


- Make sure that the connector is completely inserted to avoid air leakage.

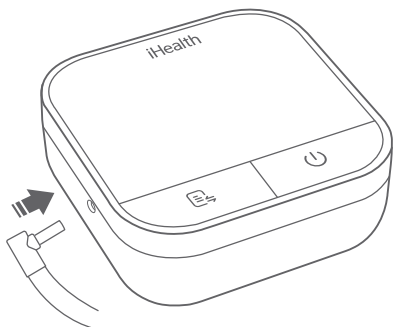

## 5 APPLYING THE CUFF

---


1. Pull the cuff through the metal loop. Slide your bare arm through the cuff and tighten it securely. Close the velcro to fasten the cuff in place.

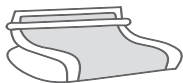

2. Ensure the bottom of the cuff is positioned  $\frac{1}{2}$ " (1 to 2 cm) above your elbow joint and fits comfortably, yet snugly, around your arm.
  3. You should be able to insert one finger between your arm and the cuff.
- 


## Left Arm

Position the cuff so that the cuff tube is in the middle of your arm and line with your middle finger.

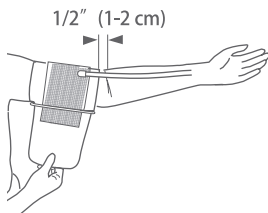

## Right Arm

Position the cuff so that the cuff tube is at the side of your elbow and in line with your little finger.

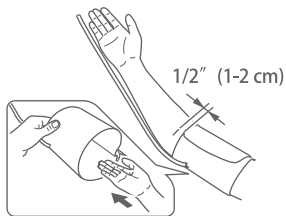

### Not sure which arm to use?

According to the 2017 AHA/ACC Hypertension Guidelines, measure both arms separately at first. If there is a significant difference in blood pressure readings, always use the arm with the higher measurement result for consistency.


#### Note:

- Please refer to the cuff circumference range in “SPECIFICATIONS” to ensure appropriate usage.
  - Always measure on the same arm for consistency.
  - Stay still and rest for 5 minutes before taking the measurement.
  - Do not apply the cuff if the arm has inflammation, acute diseases, or skin wounds.
- ⚠ To prevent measurement failure or injury, please avoid squeezing or bending the rubber tube during the measurement process.

## 6 BODY POSTURE DURING MEASUREMENT


### Sitting During Measurement

1. Sit with your feet flat on the floor and avoid crossing your legs.
2. Extend your arm with your palm facing up, resting comfortably on a flat surface.
3. Position the cuff to be at the same level as your heart.

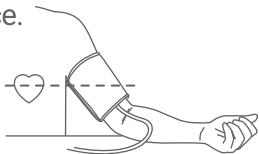


## Lying Down During Measurement


1. Lie on your back.
2. Place your left arm straight along your side with your palm facing up.
3. Ensure the cuff is positioned at the same level as your heart.

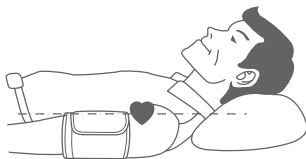

## 7 TAKING YOUR BLOOD PRESSURE READING

---






1. After securing the cuff and getting comfortable, press the  button to take a measurement.
2. The device will inflate the cuff and gradually release pressure while measuring your blood pressure.
3. Once the measurement is complete, the display will show the following;
  - Systolic and diastolic pressures (mmHg)
  - Pulse rate

- Blood pressure classification
- Irregular pulse  (if detected)

When the monitor displays the results, the indicator color will change based on the blood pressure classification. For more details, refer to “ASSESSING HIGH BLOOD PRESSURE FOR ADULTS.”



Note:



- Contact customer service if any display symbol is missing.
- If you move during the measurement, the  icon may appear, indicating a possible inaccurate result. Please measure again.
- If the  icon appears during the measurement, the cuff may be too loose or improperly wrapped. Press the  button to stop, adjust the cuff, and restart the measurement.
- At any time during a measurement, you can interrupt it by pressing the  button.
- Please consult a healthcare professional for the interpretation of the measurements.
- The device will automatically turn off after 30 seconds of inactivity. Alternatively, press the  button to turn off the device at any time.

- Verify the display matches what is shown below.





## 8 VIEWING STORED READINGS

### Latest Three Reading Averages

1. You can check the latest three measurement averages by pressing the  button.
2. When displaying the average of the last three readings, the letter  will be shown in front of the heart rate position.



## All Other Readings

1. Press the  button again to show the most recent reading.
2. When showing the records, the date of measurement, time, and the heart rate will be shown in sequence.
3. Continue pressing the  button to view the next reading.


Note:


If time and date is not set during measurement, - - : - - will be displayed.



---

## Delete All Readings

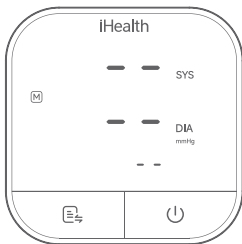

When a reading is displayed, holding the  button for seven seconds will remove all the readings stored on the device.



## 9 TRANSFERRING STORED READINGS TO THE APP

---

You may transfer your blood pressure readings to the app anytime.

1. Place your phone next to your blood pressure monitor.
2. Open the iHealth MyVitals App on your smartphone (ensure that your Bluetooth is enabled in your settings).
3. Press the  button on the monitor, and the Bluetooth symbol  will start flashing. it will stop flashing once the monitor is connected to the iHealth MyVitals app.
4. Then, the stored readings will be automatically transmitted to the app.

5. Once the readings have been successfully transferred, all stored readings on the monitor will be removed automatically.



**Note:**

The blood pressure monitor cannot connect to the iHealth MyVitals app if the battery is low, or the device is taking measurements.

## IMPORTANT SAFETY INFORMATION

1. ⚠ The device should not be used for patients with artificial hearts or lungs.
2. ⚠ The device should not be used for neonates, infants, children or persons who cannot express themselves. This device has not been validated for use on pregnant patients.
3. ⚠ The device should not be used for patients with poor peripheral circulation, noticeably low blood pressure, or low body temperature.
4. ⚠ Consult your physician before using the device for any of the

following conditions: common arrhythmias, such as atrial or ventricular premature beats or atrial fibrillation, arterial sclerosis, poor perfusion, diabetes, pre-eclampsia, and renal diseases.

5. ⚠ Do not use this device in a moving vehicle.
6. ⚠ Do not use this device If you are allergic to plastic/rubber.
7. ⚠ Please do not share the cuff to prevent the risk of infection and cross-contamination.
8. ⚠ This device can be used in conjunction with a medical AC adapter (Input: AC 100 to 240 V, 50/60 Hz, 0.2A; output: DC 5V, 1A). Do not use a cuff or AC adapter other than the ones supplied by the manufacturer. Disregarding this safety instruction may bring about a biocompatible hazard, result in measurement error, damage the device, or hurt the user.
9. ⚠ Never let children or persons incapable of expressing themselves independently use the device. Keep the device safely stored and inaccessible to children to prevent them from swallowing the batteries or other small parts.
10. ⚠ Keep the cuff tube and the cable away from children to avoid the risk of strangulation or asphyxiation.
11. ⚠ As this medical device uses an alternative small-bore connector design different from those specified in the ISO 80369 series, a misconnection can occur between it and a medical device using a different alternative small-bore connector, which can result in a HAZARDOUS SITUATION causing harm to the user. Special measures need to be taken by the user to mitigate these foreseeable risks.
12. ⚠ See the section regarding ELECTROMAGNETIC COMPATIBILITY INFORMATION for information regarding potential electromagnetic interference (EMI) or other interference between the device and other devices. (MR) Please do not use the device within the environment of the following devices: magnetic resonance imaging, computerized axial tomography, diathermy, radio frequency (RF) identification, active high-frequency surgical equipment, and electromagnetic security systems, such as metal detectors.

13. △ Stay quiet and calm, and rest for five minutes before taking your blood pressure measurement. Relax for a minimum of 1 to 1.5 minutes between measurements to allow the blood circulation in your arm to recover.
14. △ Do not speak or move your body or arm during the measurement. Motion, trembling, and shivering during measurement may affect the result.
15. △ Prolonged overinflation (cuff pressure exceeds 300 mmHg or is above 15 mmHg for longer than 3 minutes) of the bladder may cause ecchymoma (a tumor-like swelling composed of extravasated blood) of the arm.
16. △ The device might not meet its performance specifications or cause safety hazards if stored or used outside the specified temperature and humidity ranges listed in the specifications.
17. △ Consult your physician before use if any of the following scenarios are applicable:
  - 1) The cuff will be applied over a wound or inflammation disease;
  - 2) The cuff will be applied on any arm where intravascular access or therapy, or an arterio-venous (A-V) shunt, is present;
  - 3) The cuff will be applied to the arm on the same side as a mastectomy or lymph node clearance;
  - 4) The device will be used simultaneously with other monitoring medical equipment on the same arm;
18. △ Blood pressure measurements determined by this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard Institute, electronic or automated sphygmomanometers.
19. △ This equipment has been tested and found to comply with the limits for a Class B digital device under part 15 of the FCC Rules. These limits are designed to protect reasonably against harmful interference in a residential installation. This equipment generates, uses, and can radiate RF energy and, if not installed and used by the instructions, may cause harmful interference to radio communications. However,

there is no guarantee that interference will not occur in a particular installation. Suppose this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on. In that case, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and the receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

20.  $\Delta$  A signal will be displayed if the blood pressure measurement procedure detects an irregular heartbeat (IHB) brought on by common arrhythmias. Under this condition, the electronic sphygmomanometer can keep functioning, but the results may not be accurate. Please consult your physician for a precise assessment.

There are two conditions under which the signal of IHB will be displayed:

- 1) The coefficient of variation of pulse period  $>25$  percent.
- 2) The difference in adjacent pulse periods is  $\geq 0.14$  seconds, and the number of such pulses is more than 53 percent of the total number.

21.  $\Delta$  Please check the condition of the arm being used to ensure that the device is not impairing the patient's blood circulation when in use.

## ASSESSING HIGH BLOOD PRESSURE FOR ADULTS

The categories, shown below are based on the ACC/AHA 2017 High Blood Pressure Clinical Practice Guideline.

Category	Systolic(mmHg)		Diastolic(mmHg)	Color Indicator
Normal	<120	and	<80	Green
Elevated	120–129	and	<80	
High Blood Pressure (Hypertension) Stage 1	130–139	or	80 - 89	Yellow
High Blood Pressure (Hypertension) Stage 2	≥140	or	≥90	Red
HYPERTENSIVE CRISIS	>180	and/or	>120	Flashing Red

\* Individuals with Systolic BP and Diastolic BP in 2 categories should be designated to the higher BP category.

BP indicates blood pressure (based on an average of  $\geq 2$  careful readings obtained on  $\geq 2$  occasions).

Note: The guidelines are not intended to be used as a basis for self diagnosis or emergency conditions but only to differentiate between general classifications of blood pressure levels.

## NORMAL BLOOD PRESSURE FLUCTUATION

Blood pressure is affected by various factors, including excitement, stress, body position, and physical activities such as eating, drinking, smoking, or even taking a blood pressure measurement. As a result, it is unusual to obtain identical blood pressure readings multiple times.

Blood pressure fluctuates constantly throughout the day and night. Typically, it continues to rise during the day and peaks while most people are awake and active. It then drops in the evening, reaching its lowest between midnight and 3 a.m. while most people sleep. Considering the above information, measuring your blood pressure at approximately the same time every day is recommended.


Taking measurements more often than necessary may cause an injury due to blood flow interference, so please always rest at least 60 to 90 seconds between measurements to allow the blood circulation in your arm to recover.

## TECHNICAL ALARM DESCRIPTION

The device will show 'HI' or 'Lo' as a technical alarm on the screen, with no delay, if the determined blood pressure (systolic or diastolic) is outside the range stated in the "SPECIFICATIONS" section. In this case, you should consult a physician or check to ensure you have followed the instructions.

The technical alarm is preset and cannot be adjusted or deactivated. The technical alarm is non-latching and does not require a reset. The signal displayed on the screen will disappear automatically after about eight seconds.

## TROUBLESHOOTING

PROBLEM	POSSIBLE CAUSE	SOLUTION
Display shows abnormal result	The cuff position was not correct or it was not properly tightened.	Apply the cuff correctly and try again.
	Body posture was not correct during testing.	Review the "BODY POSTURE DURING MEASUREMENT" section of the instructions and re-test.
	Speaking, arm or body movement, angry, excited or nervous during testing.	Re-test when calm, without speaking or moving during the test.
	Irregular heartbeat (arrhythmia).	It is not suitable for people with common arrhythmia to use this blood pressure monitor.
Display shows low battery symbol 	Low battery.	Change the batteries.

<b>PROBLEM</b>	<b>POSSIBLE CAUSE</b>	<b>SOLUTION</b>
Display shows "Er 0"	Pressure system is unstable before measurement.	Do not move and try again.
Display shows "Er 1"	Failure to detect systolic pressure.	
Display shows "Er 2"	Failure to detect diastolic pressure.	
Display shows "Er 3"	Pneumatic system blocked or cuff is too tight during inflation.	Apply the cuff correctly and try again. If the device is still abnormal, please contact the local distributor or the factory.
Display shows "Er 4"	Pneumatic system leakage or cuff is too loose during inflation.	
Display shows "Er 5"	Cuff pressure above 300 mmHg.	
Display shows "Er 6"	More than three minutes with cuff pressure above 15 mmHg.	
Display shows "Er 7"	Inner memory error.	Measure again after five minutes. If the device is still abnormal, please contact the local distributor or the factory.
Display shows "Er 8"	Device parameter checking error.	
Display shows "Er A"	Pressure sensor parameter error.	
Display shows "Er ✖"	Bluetooth connection unsuccessful, device is abnormal, or strong EMI is present.	
No response when you press button or load battery.	Incorrect operation or strong EMI.	Reset iOS/Android device. Reset device. Make sure the device and iOS/Android device are away from other electrical equipment.  Take out batteries for five minutes, then reinstall all batteries.

## MAINTENANCE

1. ⚠️ Avoid dropping or subjecting the device to substantial impacts.
2. ⚠️ Avoid high temperatures and prolonged exposure to direct sunlight. Do not immerse the device in water, which will damage it.
3. ⚠️ Changes or modifications not approved by the manufacturer will void the user's warranty. Do not disassemble or attempt to repair the device or its components.
4. ⚠️ Remove the batteries if the device is not used for longer than one month to avoid damage due to battery leakage.
5. ⚠️ It is recommended that the device's performance be checked every two years.
6. Clean the device with a dry, soft cloth or a soft cloth dampened with water, disinfectant alcohol, or diluted detergent (wring out the cloth to remove as much liquid as possible before wiping the device).
7. Please keep the cuff clean. If the cuff becomes dirty, remove it from the device and clean it by hand with mild detergent, then rinse it thoroughly with cold water. Never dry the cuff in a clothes dryer or iron it. For personal use, it is recommended that the cuff be cleaned after it has been used approximately 200 times. Disinfecting is recommended if the cuff is used in a hospital or a clinic. Wipe the cuff's inner side (the side that contacts the skin) with a soft cloth, lightly moistened with Ethyl alcohol (75 to 90 percent). Then, air-dry the cuff.
8. We can provide product circuit diagrams and repairable component information to qualified maintenance service personnel if necessary.
9. Please wait when moving the device between extreme temperatures (e.g., storage, during transport) to a normal operating environment. The device takes approximately two hours to warm up or cool down before use.
10. ⚠️ The device shall not be serviced or maintained while in use.

## EXPLANATION OF SYMBOLS ON UNIT



Symbol for “THE INSTRUCTION MANUAL MUST BE READ.” (The sign’s background color is blue, and the sign’s graphical symbol is white.)



Symbol for “TYPE BF APPLIED PARTS.” (The cuff is type BF applied part.)



Symbol for “ENVIRONMENT PROTECTION.” Electrical products should not be disposed of as household waste. Please recycle where facilities exist. Check with your local authority or retailer for advice on recycling.



Symbol for “MANUFACTURER.”



Symbol for “DATE OF MANUFACTURE.”



Symbol for “SERIAL NUMBER.”

**IP21**

The first characteristic numeral symbol for “Degrees of protection against access to hazardous parts and solid foreign objects.” The second characteristic numeral symbol is “Degrees of protection against water ingress.”



MR Unsafe



CLASS II equipment

## WARRANTY INFORMATION

iHealth Labs, Inc. ("iHealth") warrants the iHealth hardware (the "Product"), and only the Product, against defects in materials and artistry under regular use for one year from the date of purchase by the original purchaser ("Warranty Period"). Under this Limited Warranty, if a defect arises and a valid claim is received by iHealth within the Warranty Period regarding the Product, at its option and to the extent permitted by law, iHealth will either (1) repair the Product using new or refurbished replacement parts or (2) exchange the Product with a new or refurbished Product. In the event of a defect, to the extent permitted by law, these are the sole and exclusive remedies. This warranty does not apply: (a) to consumable parts, such as the cuff or the battery, that diminish over time unless failure has occurred due to a defect in materials or quality; (b) to cosmetic damage, including but not limited to scratches, dents; (c) to damage caused by accident, abuse, misuse, contact with liquid; (d) to damage caused by operating the iHealth product outside the instruction manual, the technical specifications or other iHealth product published guidelines; (e) to damage caused by service performed by anyone who is not a representative of iHealth or one of its representatives.

"iHealth" is a trademark of iHealth Labs, Inc.

Manufactured for iHealth Labs, Inc.  
880 W Maude Ave, Sunnyvale, CA 94085 USA  
+1-855-816-7705  
[www.ihealthlabs.com](http://www.ihealthlabs.com)



ANDON HEALTH CO., LTD.  
No. 3 Jinping Street, Ya An Road, Nankai District,  
Tianjin 300190, China.  
Tel: +86-22-87611660

The Bluetooth® word mark and logos are registered trademarks

owned by Bluetooth SIG, Inc., and any use of such marks by iHealth Labs, Inc. is under license.

Other trademarks and trade names are those of their respective owners.

## ELECTROMAGNETIC COMPATIBILITY INFORMATION

- This medical device meets the following essential performance requirements:
  1. Limits of the error of the cuff pressure indication.
  2. Reproducibility of the blood pressure determination.
- When EMI affects the above performance, please stop using the device.
- Use of this equipment adjacent to, or stacked with, other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they usually operate.
- Use of accessories, transducers, and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.”
- Equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the KN-550LT, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.”

Table 1 - Emission

Phenomenon	Compliance	Electromagnetic Environment
RF emissions conducted emissions	CISPR 11 User 1, Class B	Home healthcare environment
Harmonic distortion	IEC 61000-3-2 Class A	
Voltage fluctuations and flicker	IEC 61000-3-3 Compliance	

Table 2 – Enclosure Port

Phenomenon	Basic EMC Standard	Immunity Test Levels
		Home Healthcare Environment
Electrostatic Discharge	IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Radiated RF EM field	IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz 80% AM at 1 kHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Refer to Table 3.
Rated power frequency magnetic fields	IEC 61000-4-8	30 A/m 50 Hz or 60 Hz
Proximity magnetic fields	IEC 61000-4-39	Refer to Table 5.

Table 3 – Proximity Fields from RF Wireless Communications Equipment

Test frequency (MHz)	Band (MHz)	Immunity Test Levels
		Professional healthcare facility environment
385	380-390	Pulse modulation 18 Hz, 27 V/m
450	430-470	FM, ±5 kHz deviation, 1 kHz sine, 28 V/m
710	704-787	Pulse modulation 217 Hz, 9 V/m
745		
780		
810	800-960	Pulse modulation 18 Hz, 28 V/m
870		
930		
1720	1700-1990	Pulse modulation 217 Hz, 28 V/m
1845		
1970		
2450	2400-2570	Pulse modulation 217 Hz, 28 V/m
5240	5100-5800	Pulse modulation 217 Hz, 9 V/m
5500		
5785		

Table 4 – Input AC Power Port

Phenomenon	Basic EMC Standard	Immunity Test Levels
		Home Healthcare Environment
Electrical fast transients/ burst	IEC 61000-4-4	±2 kV 100 kHz repetition frequency
Surges (line-to-line)	IEC 61000-4-5	±0.5 kV, ±1 kV
Conducted disturbances induced by RF fields	IEC 61000-4-6	3 V, 0.15 MHz to 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz
Voltage dips	IEC 61000-4-11	0% U <sub>r</sub> ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°
		0% U <sub>r</sub> ; 1 cycle and 70% U <sub>r</sub> ; 25/30 cycles Single phase: at 0°
Voltage interruptions	IEC 61000-4-11	0% U <sub>r</sub> ; 250/300 cycles

Table 5 - Test Specifications for Enclosure Port Immunity to Proximity Magnetic Fields

Test Frequency	Modulation	Immunity Test Level (A/m)
30 kHz	CW	8
134, 2 kHz	Pulse modulation 2,1 kHz	65
13, 56 MHz	Pulse modulation 50 kHz	7.5

Date of issue: May 12, 2025

# iHealth

## iHealth Labs, Inc.

880 W Maude Ave, Sunnyvale, CA 94085 USA

Phone: +1-855-816-7705

---



Scan to  
Download APP



For more information,  
please visit:  
[www.ihealthlabs.com](http://www.ihealthlabs.com)